



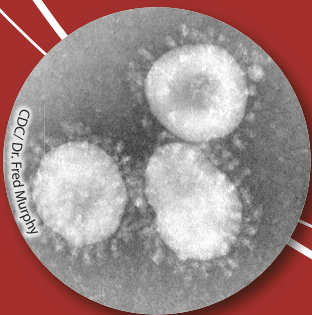

Quatro Vet Uni-Strip


« Ready to use » immunochromatographic kit for detection of Rotavirus, *Cryptosporidium*, Coronavirus and *Escherichia coli* K99 in bovine stool specimens

- Specific and sensitive
- Easy to perform
- Convenient
- Simple and accurate
- No special skill for interpretation
- Minimal faecal manipulation
- No instrumentation

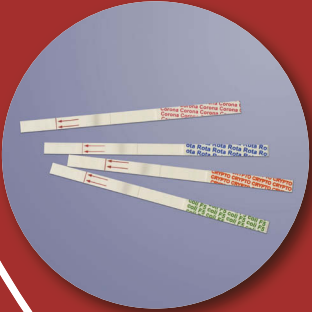

www.corisbio.com

Quatro Vet Uni-Strip

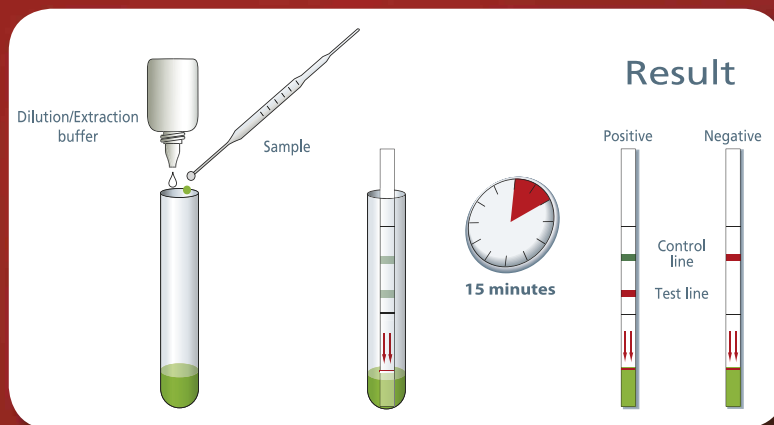

Performances of the test

Rotavirus test:

Results on 67 faecal samples.
Sensitivity: 92.85% - Specificity: 100%
Reliability: 98.5%

ELISA / Rota	Positive	Negative	Total
Positive	13	0	13
Negative	1	53	54
Total	14	53	67

Escherichia coli K99 test:

Results on 42 faecal samples.
Sensitivity: 100% - Specificity: 94.11%
Reliability: 95.23%

ELISA / E. coli K99	Positive	Negative	Total
Positive	8	2	10
Negative	0	32	32
Total	8	34	42

Cryptosporidium test:

Results on 113 faecal samples.
Sensitivity: 94.74% - Specificity: 92%
Reliability: 92.9%

ELISA / Crypto	Positive	Negative	Total
Positive	36	6	42
Negative	2	69	71
Total	38	75	113

Coronavirus test:

Results on 69 faecal samples.
Sensitivity: 92.59% - Specificity: 100%
Reliability: 97.10%

ELISA / Corona	Positive	Negative	Total
Positive	25	0	25
Negative	2	42	44
Total	27	42	69

For more technical information

client.care@corisbio.com


Ordering Information

Product Name	Description	Storage	Code
Quatro Vet Uni-Strip	10 individual strips with 4 tests, buffers, sample taking material (test tubes, loops, tube holder)	4°C to 30°C	C-1540

All diagnostics available in Coris BioConcept catalogue :

Adenovirus, Adenovirus (Respiratory), Adenovirus 40 and 41, *Clostridium difficile*, Bovine Coronavirus, *Cryptosporidium*, *Escherichia coli* O157, *Escherichia coli* K99, *Giardia*, *Helicobacter pylori*, Influenza A and B viruses, *Legionella pneumophila*, Respiratory Syncytial Virus, Rotavirus. Mefloquine, Proguanil. *Leishmania spp.*, *Trypanosoma cruzi*, *P. aeruginosa* efflux.

Distributed by:

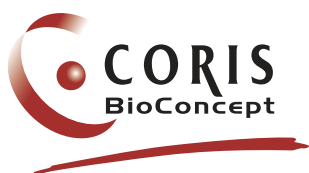

Science Park CREALYS
Rue Jean Sonet 4A - 5032 Gembloux - Belgium
Ph.: +32 (0) 81 719 910 - Fx.: +32 (0) 81 719 919
sales@corisbio.com - www.corisbio.com

MADE IN BELGIUM

Fo 7.5-32-06/07
January 2012