


The original USA made

# Little Sucker<sup>®</sup> Nasal Tip

NASAL SUCTION DEVICE

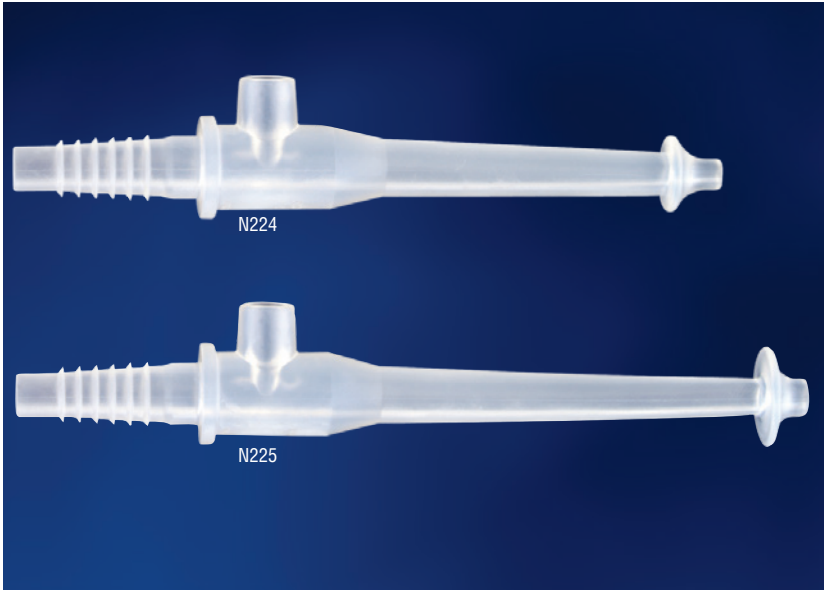

## FEATURES & BENEFITS

- Specialized tip designed to minimize potential tissue trauma
- Designed for single handed suctioning
- Thumb port for intermittent suctioning
- Soft, flexible tip similar to a bulb syringe
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

CAT. #	ITEM	QTY/UNIT
N224	Preemie Nasal Tip	50/box
N224TP	Preemie 2-piece	50/box
N225	Standard Nasal Tip	50/box
N225TP	Standard 2-piece	50/box



U.S. Patent  
#7,635,361; # D668,757;  
#6,958,050; #D474,269



# Little Sucker® Nasal Tip

## NASAL SUCTION DEVICE

### INDICATIONS FOR USE

The Little Sucker Nasal Tip is indicated for use as a tip for suction tubing to aspirate fluids from the nares.

### DIRECTIONS FOR USE

- 1 Connect barbed end of Little Sucker Nasal Tip to suction line.
- 2 Set suction to appropriate pressure per hospital protocol or physician recommendation.
- 3 Block thumb port with thumb to begin suctioning.
- 4 Suction nares with intermittent suction. Remove thumb from thumb port to stop suctioning. Block thumb port again to continue suctioning.
- 5 Suction sterile water through Little Sucker Nasal Tip after each use to clear debris.
- 6 Replace Little Sucker Nasal Tip every 24 hours or per hospital protocol, whichever is sooner.

### GENERAL INFORMATION

#### Product Materials

Little Sucker: TPE (thermoplastic elastomer)  
This product is not made with natural rubber latex or plasticizer DEHP.

#### Size

N224 Preemie  
Length: 106 mm  
Tip opening diameter: 2 mm  
Nasal guard diameter: 11 mm  
Thumb port opening diameter: 5 mm

N224TP Preemie Length: 109 mm  
Tip opening diameter: 2 mm  
Nasal guard diameter: 11 mm  
Thumb port opening diameter: 6 mm

N225 Standard  
Length: 120 mm  
Tip opening diameter: 3 mm  
Nasal guard diameter: 15 mm  
Thumb port opening diameter: 5 mm

N225TP Standard  
Length: 109 mm  
Tip opening diameter: 2 mm  
Nasal guard diameter: 15 mm  
Thumb port opening diameter: 6 mm

#### Packaging

Individually packaged: Yes  
Package type: Polyethylene bag  
Dispenser box: Yes

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Boxes are printed with environmentally friendly ink on recyclable material.*

### CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

For nasal use only.

Inserting the Little Sucker Nasal Tip too far into the nares can cause tissue trauma.

The Little Sucker Nasal Tip should remain free of debris.

Single patient use only.



**For more information or to order your FREE sample call  
1-800-966-0500 or visit [neotechproducts.com](http://neotechproducts.com)**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

