


Neotech RAM Cannula[®]

NASAL OXYGEN CANNULA

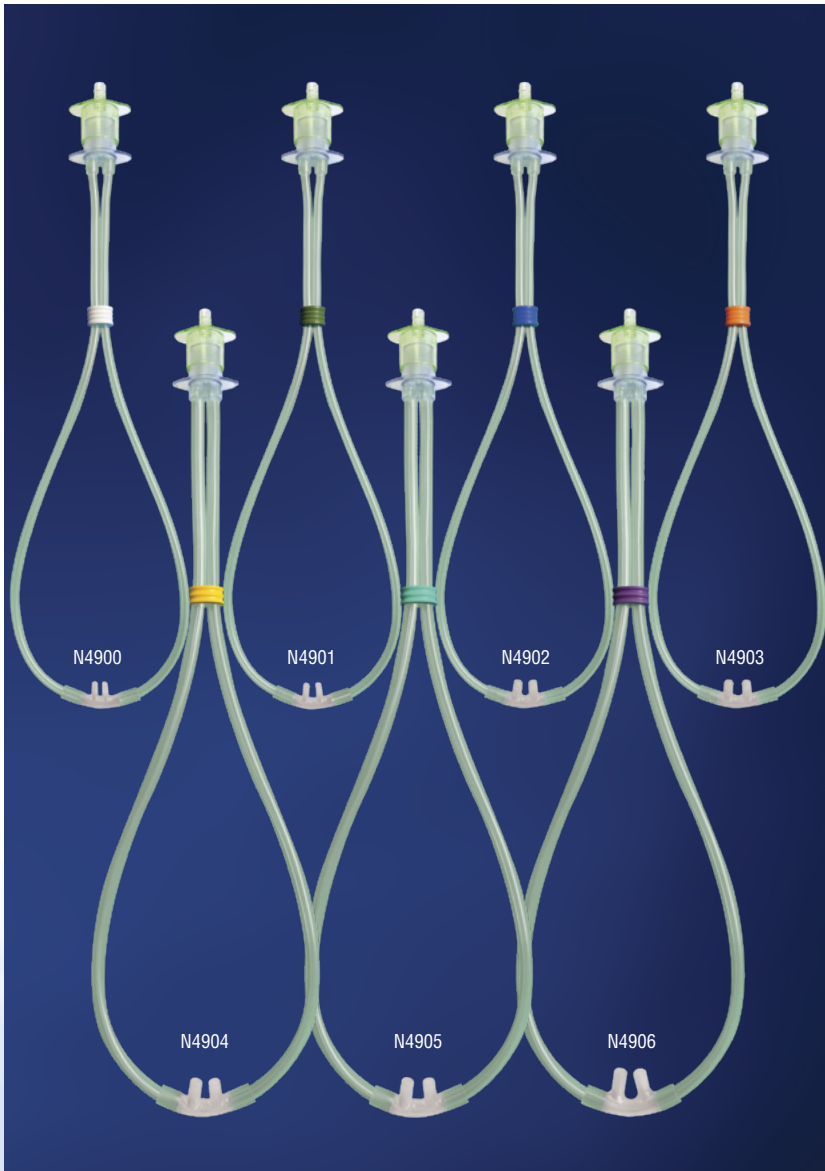

FEATURES & BENEFITS

- For use with low/high flow humidified oxygen
- Soft, curved prongs
- Color coded sizes
- 15 mm oxygen tubing adapter included
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

CAT. #	ITEM	QTY/UNIT
N4900	White	10/box
N4901	Green	10/box
N4902	Blue	10/box
N4903	Orange	10/box
N4904	Yellow	10/box
N4905	Teal	10/box
N4906	Purple	10/box
N4820	Oxygen Tubing Adapter	50/box



Made in USA



Clinician Invented



U.S. Patent #8,353,296; #D681,193; #9,027,562; #9,808,593; #9,861,777; Additional Foreign Patents Pending

Neotech RAM Cannula®

NASAL OXYGEN CANNULA

INDICATIONS FOR USE

The Neotech RAM Cannula is intended to be used for preterm and term neonates, infants, and pediatric patients who require supplemental oxygen, respiratory support or assistance in breathing in an ambulatory, hospital or institutional environment. It is used to provide oxygen with the following methods:

1. Heated or unheated, humidified low flow oxygen.
2. Heated, humidified high flow oxygen.

DIRECTIONS FOR USE

Application

- 1 Select proper size cannula by using RAM size guide provided in package. Ideal prongs size will fill approximately 80% of nares. Ensure that prongs do not fill nares completely.
- 2 Attach 15 mm oxygen tubing adapter (provided) to RAM cannula. Attach cannula to oxygen source. Ensure all cannula connections are properly secured.
- 3 Open and test the level of oxygen flow before putting the device on the patient.
- 4 Insert prongs into patient's nares and allow a small gap between patient's septum and base of prongs.
- 5 Secure cannula to face with EZ-Hold® Tubing and Cannula Holder or comparable product.
- 6 Replace Neotech RAM Cannula every 14 days or per hospital protocol, whichever is sooner. Dispose of used RAM Cannula per hospital protocol.

GENERAL INFORMATION

Product Materials

Cannula tube: PVC
Cannula prongs: PVC
15mm adapter: PVC
O-ring: Silicone
Oxygen adapter: Acrylic
This product is not made with natural rubber latex or plasticizer DEHP.

Size

N4900 White
Nasal prongs septal space: 2.5 mm
Nasal prongs outer diameter: 3 mm
N4901 Green
Nasal prongs septal space: 4.25 mm
Nasal prongs outer diameter: 3 mm
N4902 Blue
Nasal prongs septal space: 4.75 mm
Nasal prongs outer diameter: 3.5 mm

N4903 Orange

Nasal prongs septal space: 5 mm
Nasal prongs outer diameter: 4 mm

N4904 Yellow

Nasal prongs septal space: 5.75 mm
Nasal prongs outer diameter: 5.25 mm

N4905 Teal

Nasal prongs septal space: 6.75 mm
Nasal prongs outer diameter: 5.75 mm

N4906 Purple

Nasal prongs septal space: 7.75 mm
Nasal prongs outer diameter: 6.5 mm

Packaging

Individually packaged: Yes

Package type: Polyethylene bag

Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information.

Boxes are printed with environmentally friendly ink on recyclable material.

CONTRAINDICATIONS, CAUTIONS & WARNINGS

The Neotech RAM Cannula is contraindicated for patients with nasal atresia or patients with facial structure deformities that prohibit adequate respiratory support.

Federal law restricts this device to sale by or on the order of a physician.

This device should only be used while the patient is under the continuous, direct supervision of healthcare professional and frequent observation of prongs position in patient's nares is necessary.

Improper selection of size, improper positioning or improper use may result in septal trauma or necrosis.

Cannula tubing can pose a potential strangulation hazard. Do not use the O-ring to secure the device to the patient.

Always begin gas flow prior to inserting prongs into patient's nares.

Do not use if package is damaged or open.

Do not soak, wash or sterilize.

Reuse of this device could result in cross contamination.

Discontinue immediately if skin irritation occurs.

Single patient use only.

Flow Resistance

N4900, N4901, N4902, N4903

0.446 cmH₂O / L / min @ 2.5 LPM

N4904, N4905, N4906

0.230 cmH₂O / L / min @ 15 LPM



2797

For more information or to order your **FREE** sample call
1-800-966-0500 or visit neotechproducts.com

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

EMERGO EUROPE
Westervoortsedijk 60
6827 AT, Arnhem,
The Netherlands