


NeoSeal[®]

NASAL PRONGS SEAL

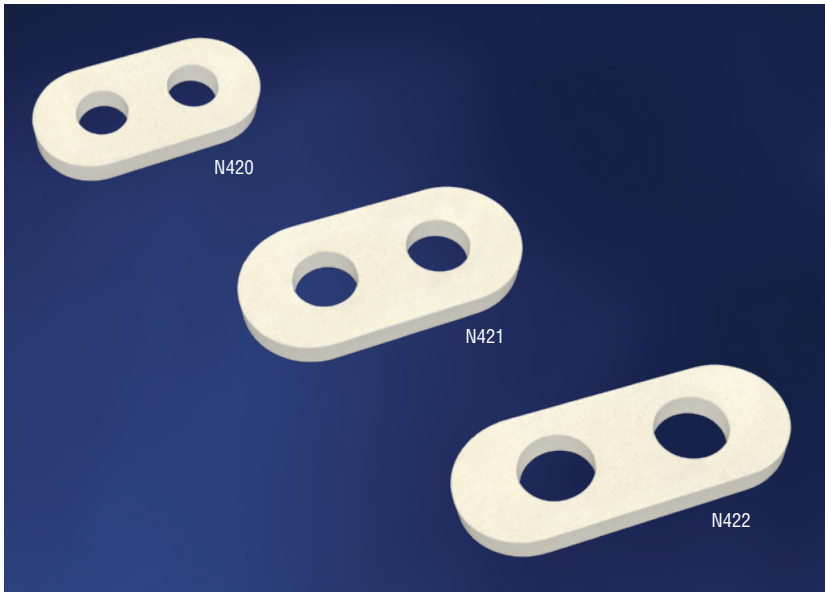

FEATURES & BENEFITS

- Designed to improve the nasal seal for more effective respiratory support
- Protects the skin of the nasal septum and nares
- Compatible with most nasal prongs
- Not made with natural rubber latex or plasticizer DEHP

CAT. #	SIZE	QTY/UNIT
N420	Micro	150/box
N421	Small	150/box
N422	Large	150/box



Patent Pending



NeoSeal®

NASAL PRONGS SEAL

INDICATIONS FOR USE

The NeoSeal is intended to create a seal around the nares and protect the skin of the nasal septum and nares. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

- 1 Peel NeoSeal from plastic liner.
- 2 Slip over nasal prongs with adhesive facing cannula.
Note: Soft, smooth side without adhesive should contact patient's skin.
- 3 Insert prongs per hospital protocol. Document placement of the NeoSeal in the patient's medical record.
Note: To minimize risk of aspiration/ingestion, ensure the nasal cannula prongs and/or NeoSeal do not enter the infant's mouth.
- 4 Replace NeoSeal daily or per hospital protocol, whichever is sooner.
Note: Replace immediately if soiled or wet.
- 5 Document removal and/or replacement in the patient's medical record.

GENERAL INFORMATION

Product Materials

PVC foam, PSA adhesive (does not contact skin)
This product is not made with natural rubber latex or plasticizer DEHP.

Size

N420 Micro: 20 mm x 10 mm
Prong opening diameter: 4.5 mm
Septal space: 4 mm

N421 Small: 25 mm x 12.5 mm
Prong opening diameter: 5.5 mm
Septal space: 5 mm

N422 Large: 30.5 mm x 12.5 mm
Prong opening diameter: 6.5 mm
Septal space: 6 mm

Packaging

Dispenser box: Yes
Perforated liner

Visit neotechproducts.com for the most up-to-date information.

Boxes are printed with environmentally friendly ink on recyclable material.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

Choking hazard. NeoSeal should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals.

Risk of ingestion or aspiration. When in use, ensure the nasal cannula prongs and/or NeoSeal do not enter the infant's mouth.

Replace immediately if soiled or wet.

Discontinue use immediately if skin irritation occurs.

Single patient use only.



For more information or to order your **FREE** sample call
1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

