


EZ-Hold[®]

TUBING AND CANNULA HOLDER

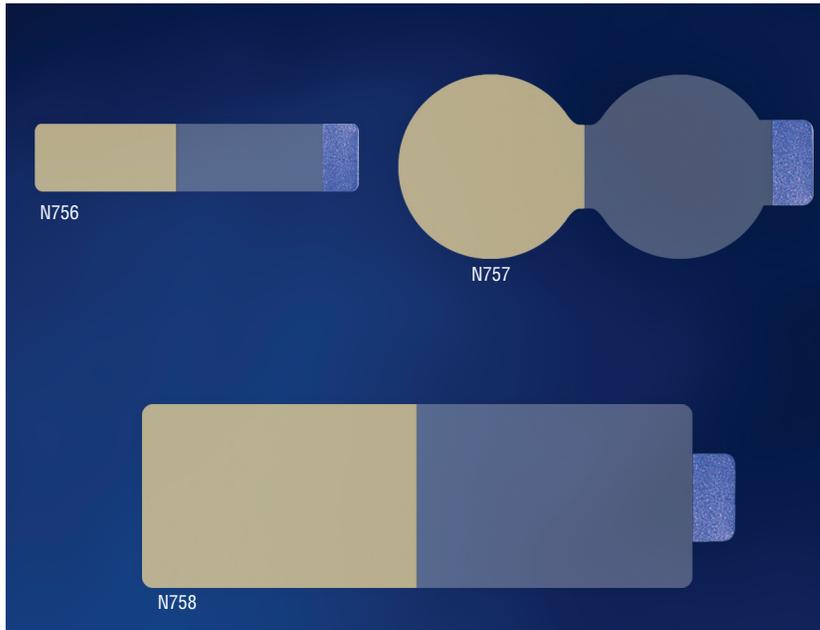

FEATURES & BENEFITS

- Skin friendly NeoBond[®] Hydrocolloid adhesive base
- Fold over flap engineered to relieve stress on tubes
- Flap designed to be opened and closed repeatedly
- Clear flap allows for better visibility
- Mini size and shape ideal for micro preemies
- Not made with natural rubber latex or plasticizer DEHP

CAT. #	ITEM	QTY/UNIT
N756	Mini	100/box
N757	Standard	100/box
N758	Extra Large	100/box



U.S. Patent #D506,547



EZ-Hold®

TUBING AND CANNULA HOLDER

INDICATIONS FOR USE

The EZ-Hold is intended to anchor and support breathing tubes.

DIRECTIONS FOR USE

Preparation

Clean and dry skin where EZ-Hold will be placed per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. EZ-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

- 1 Before removing plastic liner, hydrocolloid adhesive must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heel warmer for 10-15 seconds.
- 2 Discard white, paper liner, then peel and discard clear, printed liner from hydrocolloid base.
- 3 Apply base to skin and hold in place for 60 seconds to ensure proper adhesion.
- 4 Peel and discard second frosted liner and fold flap over tubing.

Note: Lift flap as needed to reposition or remove tubing.



Removal

- 1 Replace EZ-Hold every 5 days or per hospital protocol, whichever is sooner.
- 2 Lift flap to remove tubing.
- 3 Saturate EZ-Hold with water or saline.
- 4 Slowly peel EZ-Hold away from skin as you swab the skin with water or saline.



GENERAL INFORMATION

Product Materials

NeoBond Hydrocolloid, PET film, acrylic pressure sensitive adhesive (does not contact skin), rubber pressure sensitive adhesive (does not contact skin). This product is not made with natural rubber latex or plasticizer DEHP.

Size

N756 Mini closed: 13 mm x 25 mm
N757 Standard closed diameter: 25 mm
N758 XL closed: 25 mm x 33 mm

Packaging

Standard box, no dispense feature

Visit neotechproducts.com for the most up-to-date information.

Boxes are printed with environmentally friendly ink on recyclable material.

CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

EZ-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.



For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

