

Dale Abdominal Binder with Exclusive Stretch Material

The Dale Abdominal Binder supports abdominal muscles while encouraging mobility and promoting muscle redevelopment.

The binder is made entirely of Dale's Exclusive Stretch Material, which permits hook and loop closure at any point for more precise, more comfortable fit. Evenly distributed compression promotes coughing and deep breathing. Patients can walk more comfortably, reducing the possibility of sedentary complications such as atelectasis, pneumonia and phlebitis.

Applications

The Dale Abdominal Binder provides support and comfort following procedures such as:

- Cesarean Section
- Hysterectomy
- Liposuction
- Gastroplasty
- Abdominoplasty
- Childbirth

Key Features

- **Not made with natural rubber latex.** Eliminates concerns about patient or caregiver hypersensitivity to Latex.
- **New innovative joining stitch.** Gap-free stitching provides greater patient comfort.
- **Evenly distributed compression.** Dale's Exclusive Stretch Material distributes compression evenly around the body without riding, rolling or constricting.
- **Longer product life.** Material can be engaged and disengaged an indefinite number of times. Some fuzziness may appear but this actually improves holding power.
- **Hook and loop closure.** Binder opens and closes quickly and easily for dressing and wound inspection. Can be attached at any point on the circumference of binder for precise fit.
- **Easy-Grip Strip.** Holds up to four drainage bulbs in place on outside of abdominal binder.

Ordering Information

Dale Abdominal Binders are available from hospital/medical supply dealers, packaged one per box.

To find size, measure female hips, male waist.

H8410 410 1	3 panel	9" stretches to fit 30-45" 76-114 cm
H8410 411 1	3 panel	9" stretches to fit 46-62" 117-157 cm
H8410 418 1	3 panel	9" stretches to fit 60-75" 152-191 cm
H8410 810 1	4 panel	12" stretches to fit 30-45" 76-114 cm
H8410 811 1	4 panel	12" stretches to fit 46-62" 117-157 cm
H8410 818 1	4 panel	12" stretches to fit 60-75" 152-191 cm
H8410 820 1	4 panel	12" stretches to fit 72-94" 183-239 cm
H8410 920 1	5 panel	15" stretches to fit 72-94" 183-239 cm



Dale® Medical Products, Inc.
7 Cross Street, P.O. Box 1556
Plainville, MA 02762-0556
T: 508-695-9316 F: 508-695-6587
www.thinkdale.com

EC REP **MDSS GmbH**
Schiffgraben 41
D-30175 Hannover, Germany



Dale® is a registered trademark of Dale Medical Products in the USA and EU. EasyGrip™ is a trademark of Dale Medical Products.

US Patent # 8,066,657
©2013 Dale Medical Products, Inc.
942164-ART_REV_H

Free EasyGrip™ Strip



Dale® Abdominal Binder

with Exclusive Stretch Material




Applying the Dale® Abdominal Binder


Dale Abdominal Binders are available in a range of sizes. Two binders can be joined to fit an unusually large patient.



1. Place the binder under the patient and stretch one end out. Proper stretch provides necessary compression.

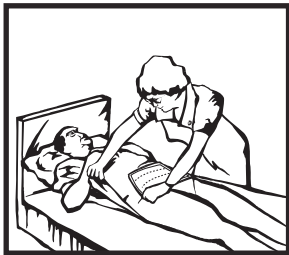

2. Stretch the binder over the patient's stomach and hold with one hand. The binder may be placed straight, or at an angle if additional pressure is needed on the upper or lower abdomen.

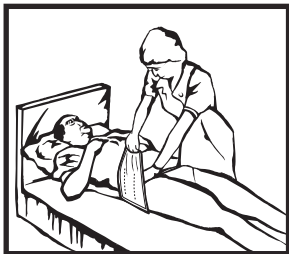

3. Stretch the other end over at the desired angle.




4. Press the binder closed while stretching.


Be sure to stretch the binder adequately. Remember, a secure fit won't restrict breathing; the stretching and compression actually facilitates deep breathing.

Using Drainage Tubes


A hole can be cut in the Dale Abdominal Binder to accommodate tubes such as draining and feeding. Place the binder on the patient so that there is only a single thickness over the draining site. Mark an X on the fabric where the hole should be made, crease the fabric and make an elliptical cut in the material (see illustration). When the fabric is stretched, the elliptical cut will become around hole to accommodate the drain. The diameter of the hole will be equal to the length of the cut. See back cover for the EasyGrip Strip Applications.

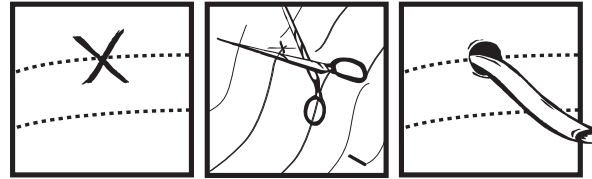


Self Application

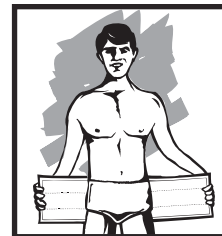

1. Position top edge just below navel and stretch out sideways.

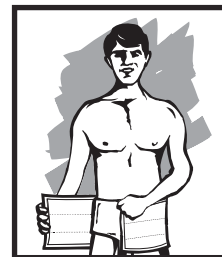


2. Wrap around.

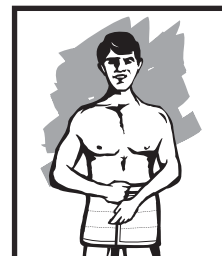

3. Press closed to secure.

†Applying the EasyGrip Strip



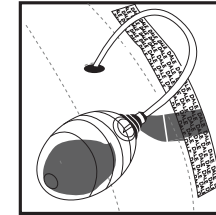

1. Lift EasyGrip Strip off Binder



For drainage bulb use, thread EasyGrip Strip through drainage bulb tab; reattach to binder

† Always follow best practice standards for drainage devices.

Laundry and Care

-  Machine wash COLD
-  Or hand wash
-  Do not bleach
-  Line dry

Machine or hand wash in cold water with mild detergent. Before washing, make sure that the hook and loop surfaces are securely fastened to prevent lint accumulation, which can interfere with the fibers and weaken their locking properties. Hang to dry.