


Dale Stabilock Endotracheal Tube Holder

For Comfortable and Secure Stabilization

Product Description

Helps prevent accidental extubation.

Application Instructions

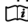-  To prevent skin ulcerations or skin tears, assess patient skin and medical conditions prior to applying. Monitor daily or more frequently.

Removal: Adhesive should be removed with care to avoid skin trauma.

- Clean tube. Let dry.
- Clean skin. Apply skin prep. Remove liners from Adhesive Base and place directly above patient's upper lip. **(A)** Trim as necessary. Adhesive Base can be applied with rounded tabs up or down.
For patients with trauma or facial hair, modify Adhesive Base for individual needs: apply to lower jaw with Rounded tabs facing up; or cut rounded tabs off and apply to patient's cheeks.
Change Adhesive Base after 3 days or sooner if necessary.
- Press neckband onto the VELCRO® portion of the base, making sure the tubing Channel is aligned with the endotracheal tube. **(B)**
- Open the fastener tabs located at both ends of the neckband and wrap the band around the head just below the ears. Secure the fastener tabs to the outside of the neckband. **(C)**
- Press the Channel down onto the endotracheal tube. **(D)**
- Just beyond the endotracheal tube's position marking, tightly wrap strong adhesive tape (such as cloth) in a spiral motion around the Channel and tube to secure. **Be sure to wrap tape onto the endotracheal tube a minimum of 2.5 cm (1") beyond the Channel. (E)** Change tape as necessary.

Ensure neckband security by readjusting the fastener tabs as necessary.

Tube Repositioning: Open fastener tabs. Hold Adhesive Base down while peeling neckband from base. Reposition endotracheal tube. Press back onto base and secure fastener tabs.

Dale strongly recommends repositioning the endotracheal tube often to help prevent injury to the lips and underlying tissues due to unrelieved pressure.

Dale strongly recommends supporting the ventilator tubing to reduce pressure on the patient's face.

Convenient Checklist for Successful Application

- ✓ **Prep** the skin every time the Adhesive Base is applied
- ✓ **Position** the Adhesive Base directly above the lip line
- ✓ **Tightly spiral wrap** strong adhesive tape around the Channel and tube
- ✓ **Be sure to wrap** tape onto the tube at least 2.5 cm (1") beyond the Channel
- ✓ **Remove** Adhesive Base gently to minimize skin damage
- ✓ **Support** ventilator tubing

For an in-service demonstration, application video and more information, go to www.dalemed.com.

Ordering Information

#H84102701 **Dale Stabilock Endotracheal Tube Holder**
Fits endotracheal tube sizes 7.0-10.0mm.
Individually packaged in box of 10.

#H84102731 **Dale Stabilock Adhesive Base**
Individually packaged in box of 10.

Disposable. For single patient use. Patent pending.
Complies with ACLS guidelines.

Dale® medical products are available from your preferred hospital/medical supply dealer.




©2010 Dale® Medical Products, Inc. VELCRO® is a registered trademark of Velcro USA, Inc.

All rights reserved.


REV D pk-270 INT




(A)

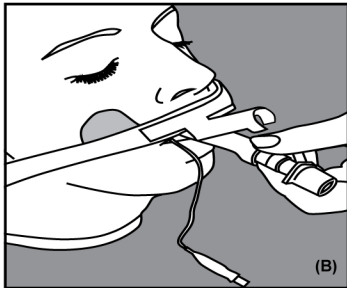


(B)

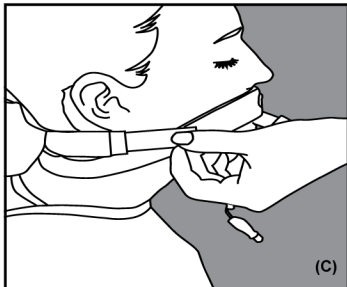

(C)

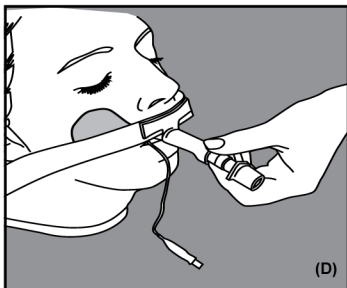

(D)

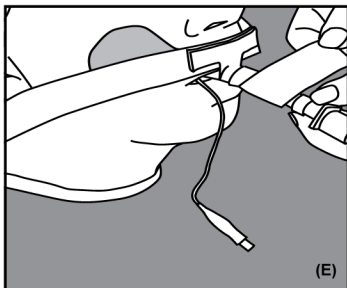

(E)