

Dale®


Tracheostomy Tube Holder

With Moisture Repellent Neckband

Product Description

Provides secure positioning of the tracheostomy tube. Superior fastener tabs are quickly and easily threaded through the flange of the tracheostomy tube. Two-piece neckband includes moisture repellent lining.

Application Instructions

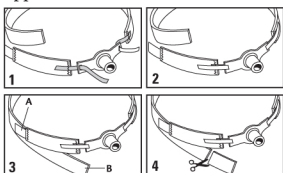

1. Thread long, narrow fastener tabs through flanges on trach plate.
2. Bring tabs back over flange and adhere to the neckband.
3. Secure fastener tab of short section (A) to the long neckband section (B). Adjust for proper length or compression.
4. Trim excess material (B only) for proper length.

Caution

Use in supervised care settings only.
Monitor to prevent unintentional disengagement.
Do not trim fastener hook tabs.
For single patient use.
Change the holder daily or more frequently as required. Change after patient bathing.
Disposable.
Do not wash.

Ordering Information

- #H84102401 Dale® 240 Blue® Tracheostomy Tube Holder
HCPCS Code: A7526
Fits most, contains a portion of stretch material to allow for edema or cough reflex. Individually packaged. Sold in boxes of 10.
- #H84102411 Dale® 241 PediPrints® Tracheostomy Tube Holder
HCPCS Code: A7526
Fits up to 18" neck.
Individually packaged. Sold in boxes of 10.
- #H84102421 Dale® 242 PediPrints® Tracheostomy Tube Holder
HCPCS Code: A7526
Fits up to 9" neck
Individually packaged. Sold in boxes of 10.

Dale medical products are available from your preferred hospital/medical supply dealer.



Latex Free



Dale® Medical Products, Inc.
7 Cross Street, P.O. Box 1556
Plainville, MA 02762-0556, USA
TEL. USA: 1-800-343-3980
TEL. INT'L: +1-508-695-9316
FAX: +1-508-695-6587
www.dalemed.com



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany



Rev D PK-Trach Insert
© Copyright 2011

(Česky, Dansk, Nederlands, English, Suomi,
Français, Deutsch, Ελληνικά, Magyar, Italiano,
日本語, 한국어, Norsk, Português,
Español, Svenska, ไทย, Türkçe)