

# EuroTest®

where quality meets economy



**EuroTest®** is a ready-to-use Bowie & Dick test pack manufactured by Sterlab based on the know-how developed from Bodipack and Quickise range.

Conscious of budget constraints in the health care system, we have imagined and produced the **EuroTest®** by cutting down as far as possible extra costs, while keeping the high grade quality standards in which Sterlab products are well known, to fit your expectations where excellence is your first criteria.

**EuroTest®** is to be used as other Bowie & Dick test packs. As soon as the cycle is over, open the pack and take out the test sheet from the pack to write down the traceability elements in the appropriate boxes.

## INTERPRETATION OF RESULTS

### Pass



The test is validated if the reactive ink (yellow before use) has turned to a homogeneous black color.

### Fail



The test cannot be validated if a discolouration (yellow/brown) appears on the test sheet (normally in the center).

REF 303  
Box of 50 units  
Carton of 4 Boxes



ISO 11140-4

