

# NeoShades® with Tabs

## PHOTOTHERAPY EYE SHIELDS



N724



N725



N726

### FEATURES & BENEFITS

- Soft, padded eye shield with cute sunglasses design
- Ocular pockets designed for patient comfort and safety (N724, N725, N726)
- Skin friendly NeoBond® Hydrocolloid tabs hold NeoShades in place
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

| CAT NO.                    | ITEM  | QTY/UNIT |
|----------------------------|-------|----------|
| N720                       | Micro | 50/box   |
| N721                       | Small | 50/box   |
| N722                       | Large | 50/box   |
| <i>With Ocular Pockets</i> |       |          |
| N724                       | Micro | 50/box   |
| N725                       | Small | 50/box   |
| N726                       | Large | 50/box   |



U.S. Patent #8,070,288;  
Additional Patents Pending



# NeoShades® with Tabs

## PHOTOTHERAPY EYE SHIELDS

### INDICATIONS FOR USE

The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.

### DIRECTIONS FOR USE

#### Preparation

- 1 Select proper size.

Approximate sizing:  
Micro – 28 weeks and under  
Small – 29-36 weeks  
Large – 37 weeks and up

- 2 Clean and dry skin on temples per hospital protocol.

*Note: It is important for skin to be as clean and dry as possible. Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.*

**DO NOT USE ALCOHOL.**



#### Application

- 1 Carefully remove hydrocolloid tabs from eye shields.

Before removing plastic liners, hydrocolloid adhesive tabs must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heel warmer for 10-15 seconds.

- 2 Peel and discard plastic liners. Place hydrocolloid adhesive tabs on skin temples and hold in place for 60 seconds to ensure proper adhesion.

- 3 Cover eyes with NeoShades and gently attach to hydrocolloid adhesive tabs.


*Note: Leave a portion of the tab exposed for easier removal.*



#### Removal

- 1 Replace NeoShades hydrocolloid adhesive tabs on skin every 5 days or per hospital protocol, whichever is sooner.
- 2 Hold edge of hydrocolloid adhesive tab in place and slowly separate NeoShades from tab.
- 3 Saturate hydrocolloid adhesive tab with water or saline.
- 4 Slowly peel hydrocolloid adhesive tab away from skin as you swab with water or saline.

Repeat for opposite side.

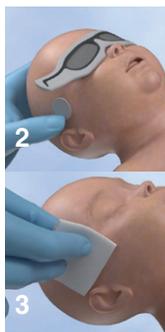

### GENERAL INFORMATION

#### Light Blocking

99.9%

#### Product Materials

NeoShades: Nylon, polyurethane  
Tabs: NeoBond Hydrocolloid, hook fabric  
This product is not made with natural rubber latex or plasticizer DEHP.

#### Size

N720, N724 Micro: 108 mm  
(fits head circumference < 26 cm)  
N721, N725 Small: 127 mm  
(fits head circumference 26-34 cm)  
N722, N726 Large: 178 mm  
(fits head circumference > 34 cm)

#### Packaging

Individually packaged: Yes  
Package type: Polyethylene bag  
Dispenser box: Yes (N720, N721, N722 only)

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Boxes are printed with environmentally friendly ink on recyclable material.*

### CONTRAINDICATIONS, CAUTIONS & WARNINGS

The NeoShades with Tabs are contraindicated for patients with facial deformation, skin infection or excoriated skin areas that would prevent the proper placement of the tabs. They are not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

Tabs should always be removed when NeoShades are taken off the patient. Do not reuse.

When using NeoShades intermittently, always assess the integrity of the hydrocolloid tabs whenever the eye shields are removed. If the hydrocolloid tabs are pulling away from the skin, remove per the directions for use and apply new hydrocolloid tabs before resuming phototherapy.

Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Discontinue use immediately if skin irritation occurs.

Single patient use only.

General transportation and storage practices are applicable.



For more information or to order your FREE sample call

**1-800-966-0500** or visit [neotechproducts.com](http://neotechproducts.com)

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355



EMERGO EUROPE  
Westervoortsedijk 60  
6827 AT, Arnhem  
The Netherlands



PPE Regulation (EU) 2016/425  
Notified Body: BSI Group (2797),  
Say Building, John M. Keynesplein 9  
1066 EP Amsterdam, Netherlands