


## NeoBar<sup>®</sup>

### ET TUBE HOLDER

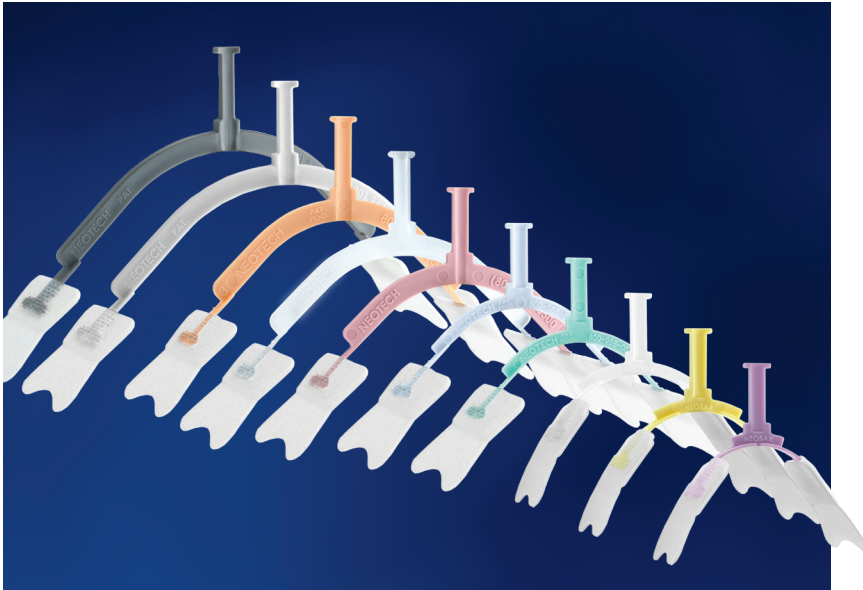

#### FEATURES & BENEFITS

- Skin friendly NeoBond<sup>®</sup> Hydrocolloid tabs
- Macro and Jumbo sizes available with medical grade acrylic adhesive
- Designed to reduce extubations, help prevent palate trauma and allow for better oral care
- Eliminates need for tape near nose or mouth
- Color coded sizes
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

CAT. #	SIZE	COLOR	QTY/UNIT
N709	Mini	Purple	5/box
N710	Ultra	Yellow	5/box
N711	Micro	White	5/box
N712	Small	Green	5/box
N713	Large	Blue	5/box
N714	XL	Rose	5/box
N715H	Macro	Clear	5/box
N716H	Jumbo	Gray	5/box
<b>With Medical Grade Acrylic Adhesive</b>			
N715F	Macro	Peach	5/box
N716F	Jumbo	Carbon	5/box
<b>Assorted</b>			
N718 Transport (1 each N709, N710, N711, N712, N713, N714)			
N719N Assorted (2 each N709, N710, N711, N712, N713, N714)			

U.S. Patent  
#D434,496; #6,050,263



# NeoBar<sup>®</sup>

## ET TUBE HOLDER

### INDICATIONS FOR USE

The NeoBar is intended to secure an endotracheal tube. It is intended for use on pediatric (neonates, infants and children) patients.

### DIRECTIONS FOR USE

#### Preparation

- 1 After patient is intubated, select proper size NeoBar by using NeoBar measuring tape strip provided.

Place mid line of strip at nasal septum and wrap to ear. Color that falls over opening of ear canal corresponds to correct NeoBar size.

*Note: if unsure between two sizes, try smaller size first.*

- 2 To ensure proper fit, position NeoBar across center of mouth, between upper and lower lips. NeoBar should not contact lips. Hydrocolloid tabs must be just in front of ear.

- 3 Clean and dry skin on cheeks per hospital protocol.

*Note: It is important for skin to be as clean and dry as possible. Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.*

DO NOT USE ALCOHOL.

#### Application

- 1 Before removing plastic liners, hydrocolloid adhesive tabs must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heel warmer for 10-15 seconds.

- 2 Peel and discard plastic liners. Apply hydrocolloid adhesive tabs in front of ear on bone and hold in place for 60 seconds to ensure proper adhesion.

*Note: Ensure ET tube is below platform to reduce pressure on palate or gums.*

- 3 Wrap 1/2" (12.5 mm) white or cloth tape completely around NeoBar platform first, then continue taping around both platform and ET tube.

#### Removal

- 1 Replace NeoBar every 5 days or per hospital protocol, whichever is sooner.
- 2 Unwrap tape to separate ET tube from NeoBar platform.
- 3 Saturate tabs with water or saline.
- 4 Slowly peel hydrocolloid adhesive tabs away from skin as you swab with water or saline.

### GENERAL INFORMATION

#### Product Materials

NeoBar: Polypropylene  
Hydrocolloid tabs: NeoBond<sup>®</sup> Hydrocolloid, non-woven fabric  
Acrylic tabs: Friendly foam, non-woven fabric, medical grade acrylic adhesive  
This product is not made with natural rubber latex or plasticizer DEHP.

#### Size

N709 Mini (Purple)  
N710 Ultra (Yellow)  
N711 Micro (White)  
N712 Small (Green)  
N713 Large (Blue)  
N714 XL (Rose)  
N715F Macro (Peach)  
N715H Macro (Clear)  
N716F Jumbo (Carbon)  
N716H Jumbo (Gray)

#### Packaging

Individually packaged: Yes  
Package type: Box  
Dispenser box: Yes

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Boxes are printed with environmentally friendly ink on recyclable material.*

### CONTRAINDICATIONS, CAUTIONS & WARNINGS

The NeoBar is contraindicated for patients with infected or excoriated skin areas that would prevent the proper placement of the tabs.

It is not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoBar should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals.

Discontinue use immediately if skin irritation occurs.

NeoBar should not contact lips.

Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.



For more information or to order your FREE sample call  
**1-800-966-0500** or visit [neotechproducts.com](http://neotechproducts.com)

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

©2022 Neotech Products LLC. Product configuration is a trademark owned by and exclusively associated with Neotech.



M552 Rev F