


tournistrip® Dispenser

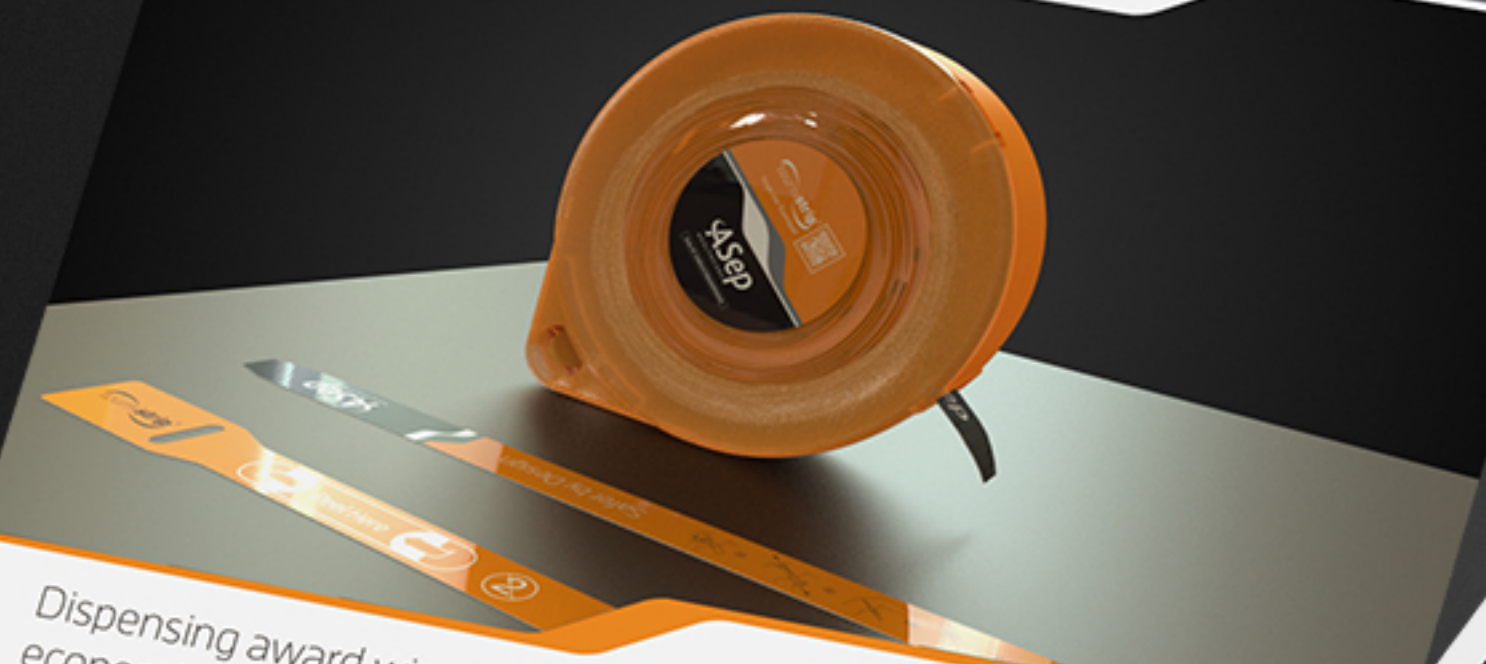

Dispensing award winning Single Use tourniquets in a safe, economical and environmentally friendly way

The tournistrip dispenser is a cleanable dispenser that can be placed on a desk, secured to a wall, or tied to a clinical trolley. The dispensing method allows the maintenance of aseptic technique. The dispenser takes the new 100 tournistrip refill pack, maintaining economy while minimising waste and environmental impact.

Tournistrip is available as refill packs containing 100 tourniquets.
Order Number: 5490LHT236/ASEPI00

Dispenser Order Number with 100 roll:
5490LHT236/ASEPI00DIS



T: 01296 663 930
E: support@asephealthcare.com
www.asephealthcare.com



tournistrip®

Why Use tournistrip?

- Single Patient Use**
Unique design and construction to aid infection prevention and prevent cross infection.
- Single handed engagement and release**
Can be released with a single hand, minimizing contact.
- Easy To Use**
Easy to tension and adjust. Can be repositioned on the same patient up to 5 times.
- Flexible**
Two tournistrips can be joined together for larger arms and bariatric patients.
- Comfort**
Tournistrip Doesn't pinch the skin like "rubber band" designs.
- Clinically Proven**
Tournistrip is the only medical tourniquet to have been clinically trialed, involving two London Teaching Hospitals with 226 patients and



Scan to see the tournistrip Video demonstration



T: 01296 663 930
E: support@asephealthcare.com
www.asephealthcare.com

Instructions For Use

- 1 Remove wrap from Refill Pack
- 2 Place strips into dispenser with cardboard core in centre
- 3 Feed tab of strip through the aperture
- 4 Close lid of dispenser and place in desired clinical area

