

Dale® Hold-n-Place™ Foley Catheter Holders/Leg Band







with Exclusive Green Locking Tabs

Product Description

Features patented green locking device that securely positions the Foley catheter on the patient. Disposable. Available as leg band or waistband.

Leg Band Application Instructions

1. Position leg band high around patient's thigh with Dale product label pointing to inside of leg. Stretch leg band in place and fasten Velcro® tab. Proper compression has been achieved when two fingers fit snugly under the band.
2. Place Foley catheter over white stitching of green tab. Leave an ample loop in catheter above leg band to avoid bladder traction; or stretch catheter for traction when required.
3. With catheter in desired position, insert the narrow, green tab over catheter and through the square opening so that green tabs overlap.
4. Lock catheter in position by pulling green tabs in opposite directions. Secure tabs in place. To reposition catheter: raise either green tab, adjust catheter, and refasten tab.
5. Another method of securement is to locate the Y port of the proximal end of the catheter into the locking device.

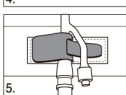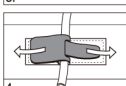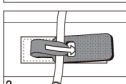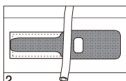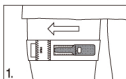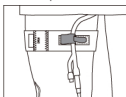

Waistband Application Instructions

1. To secure the catheter in place, position the waistband around the abdomen, above the wound site. No shaving necessary. The green catheter-locking tab should be centered over the drain site. After the band has been positioned, press locking tab onto the waistband.
2. Place catheter over white stitching of green tab.
3. Repeat steps 3, 4, and 5 above.

Additional Benefits: To help prevent meatal irritation, leg band can be rotated in either direction. Alternating legs periodically may also add to greater patient comfort.

For single patient use.

Hold-n-Place™ Ordering Information

#H84103161 **Dale® Hold-n-Place™ Catheter Holders/Leg Band**

HCPCS Code: A4334

One size fits most.
Individually packaged.
Sold in boxes of 10.

#H84103301 **Dale® X-Large Hold-n-Place™ Catheter Holders/Bariatric Leg Band**

Can also be used as waistband

HCPCS Code: A4335

Leg Band—fits most bariatric.
Waistband—one size fits most.
Individually packaged.
Sold in boxes of 10.

#H84101501 **Dale® Hold-n-Place™ Foley Catheter Holders/Adhesive Patch.**

One size fits All. Individually packaged.
Sold in boxes of 50.

Dale medical products are available from your preferred hospital/medical supply dealer.

US Patent Number 6,419,660

Velcro® is a registered trademark of VELCRO USA, Inc.

© Copyright 2009. Dale® Medical Products, Inc. USA.

All rights reserved.



Dale® Medical Products, Inc.
7 Cross Street, P.O. Box 1556
Plainville, MA 02762-0556, USA
TEL. USA: 1-800-343-3980
TEL. INT'L: +1-508-695-9316
FAX: +1-508-695-6587
www.dalemed.com



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany
Rev A pk-316/330 INT



(Česky, Dansk, Nederlands, English, Suomi,
Français, Deutsch, Ελληνικά, Magyar, Italiano,
日本語, 한국어, Norsk, Português,
Español, Svenska, ภาษาไทย, Türkçe)