


English

Dale® NasoGastric Tube Holder

Superior Security for Naso-Gastric Tubes

The Dale NasoGastric Tube Holder is ideal for holding feeding and aspiration tubes such as Salem sump tubes, Levin stomach tubes, naso-enteric feeding tubes, and naso-gastric intestinal tubes. Skin-friendly adhesive stays in place for extended periods, yet is easy to remove.

Application Instructions

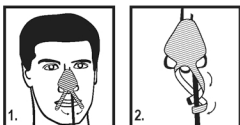

1. After tube is in place, cleanse tube and nose.
2. Peel holder completely from liner, starting at nose pad. Apply nose pad to nose (Figure 1).
3. Choose tab closest to tube and begin to tightly wrap the tube in a spiraling fashion (Figure 2).
4. Wrap remaining tab using same spiraling method (Figure 2).

To remove: Simply use the non-adherent tips to unwrap tabs from tube.

For single patient use.

Ordering Information

#H84101601 Dale® NasoGastric Tube Holder
Fits all. Individually packaged.
Sold in boxes of 50.

#H84101611 Dale® NasoGastric Tube Holder
Small. Individually packaged.
Sold in boxes of 50.



Latex-free



Dale® Medical Products, Inc.
7 Cross Street, P.O. Box 1556
Plainville, MA 02762-0556, USA
TEL. USA: 1-800-343-3980
TEL. INT'L: +1-508-695-9316
FAX: +1-508-695-6587
www.dalemed.com



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany



Rev D pk-160 INT
© Copyright 2012

(Česky, Dansk, Nederlands, English, Suomi, Français, Deutsch, Ελληνικά, Magyar, Italiano, 日本語, 한국어, Norsk, Português, Español, Svenska, ภาษาไทย, Türkçe)