


Product Number H84106201

Dale® Transducer Holder**LATEX**


Latex Free

Product Description

The Dale Transducer Holder provides a level and secure platform, which can accommodate up to three transducers, for your patients on hemodynamic monitoring.

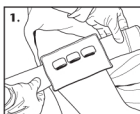**Application Instructions:**

1. Position the Holder along the phlebostatic axis (midchest) and wrap around the patient's arm. Press VELCRO® to secure.
2. Open the necessary VELCRO® tabs, thread through the slot(s) of the transducer(s) and press to secure.

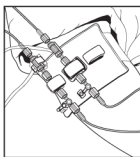

For single patient use.

Ordering information

#H84106201 Dale® Transducer Holder.
 Stabilizes up to three transducers.
 Fits most adults.
 Individually packaged.
 Sold in boxes of 10.

Dale® medical products are available from your preferred hospital/medical supply dealer.

VELCRO® is a registered trademark of Velcro USA, Inc. © Copyright 2010



Dale® Medical Products, Inc.
 7 Cross Street, P.O. Box 1556
 Plainville, MA 02762-0556, USA
 TEL. USA: 1-800-343-3980,
 TEL. INT'L: +1-508-695-9316
 FAX: +1-508-695-6587
 www.dalemed.com



MDSS GmbH
 Schiffgraben 41
 D-30175 Hannover, Germany
 Rev B pk-620 INT



(Česky, Dansk, Nederlands, English, Suomi, Français, Deutsch, Ελληνικά, Magyar, Italiano, 日本語, 한국어, Norsk, Português, Español, Svenska, ไทย, Türkçe)